

# How to Activate Test Mode on the TCL200 Alarm Clock

**ATTENTION: To activate test mode on the TCL200 alarm clock, the batteries must be removed and the vibration pad must be disconnected from the alarm clock.**

1. With the power supply disconnected from the TCL200 alarm clock, turn all the switches located on the back of the alarm clock to the right.
2. Turn the alarm ON/OFF switch to the “ON” position. The ON/OFF switch is located on the right side of the alarm clock.
3. Plug the power supply into an electrical outlet. DO NOT connect the power supply to the alarm clock at this point.
4. Press and hold the “SNOOZE” button located at the top of the alarm clock.
5. While the “SNOOZE” button is pressed, plug the power supply into the TCL200 alarm clock. Continue to press the “SNOOZE” button until 12:00 begins to flash on the screen.
6. Once 12:00 has been seen flashing on the screen, release the “SNOOZE” button. Press the “SNOOZE” button once more to activate test mode.
7. The TCL200 alarm clock alarm will sound. At this point the Volume, Signal and Pitch knobs can be adjusted. Once the adjustments have been made disconnect the power supply from the alarm clock to end the test.
8. At this point the install the batteries.
9. Connect the power supply and vibration pad.

