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**Amplicom USA Service Center**  
3018 Ambrose Avenue  
Nashville, TN 37207

V/TTY:

1-866-AMPLICOM (267-5426)

[www.amplicomusa.com](http://www.amplicomusa.com)

Distributed by: Amplicom USA LLC  
Distribuido por Amplicom USA LLC

Made in China  
Hecho en China

## User Guide

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# PTV 100

## Important Safety Instructions

When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons, including the following:

1. Do not use this product near water, for example, near a bath tub, wash bowl, kitchen sink or laundry tub, in a wet basement or near a swimming pool. 2. Do not use this product to report a gas leak in the vicinity of the leak.

### SAVE THESE INSTRUCTIONS

**Intended use** - This shaker is suitable for use on a public telephone network. Any other use is considered unintended use. Unauthorized modification or reconstruction is not permitted. **Do not** open the device and attempt any repair work yourself.

**Medical equipment** - Do not use the product near emergency or intensive care medical equipment, or during an electrical storm. If you have a pacemaker check with a medical expert before using this product.

## FCC Compliance

The FCC requires that you will be advised of certain requirements involving the use of this equipment. 1. If your device causes harm to the telephone network, the telephone company may discontinue your service temporarily. If possible, they will notify you in advance. But if advance notice is not practical, you will be notified as soon as possible. You will be advised of your right to file a complaint with the FCC. 2. If you experience trouble with the telephone, please contact the Amplicom Service Center at 1-866-AMPLICOM (267-5426) for repair/warranty information. The telephone company may ask you to disconnect this equipment from the network until the problem have been corrected or you are sure that the equipment is not malfunctioning. 3. This equipment may not be used on coin service provided by the telephone company. Connection to party lines is subject to state tariffs. (Contact your state public utility commission or corporation commission for information.) This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Privacy of communications may not be ensured when using this phone. Changes or modifications not expressly approved in writing by Amplicom may void the user's authority to operate this equipment.

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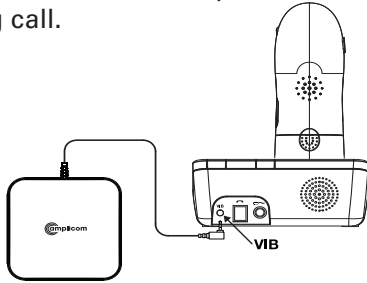
### Set up

#### Connect the shaker

When the Shaker feature on your PowerTel product is set to On, the PTV100 will vibrate each time a call is received.

- i. Connect the product as illustrated below.
- ii. Set the Shake feature on the PowerTel product to On (refer to the PowerTel phone operating instructions).
- iii. Place the PTV100 under a pillow/cushion/etc. to alert an incoming call.

## Maintenance and Warranty

### Maintenance

- Please clean your equipment's surfaces with a soft, lint-free cloth.
- Never use cleaning agents or solvents.

### Warranty

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life. The terms of the warranty do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of the warranty do not apply to the rechargeable battery pack or power packs used in the products. The period of warranty is 24 months from the date of purchase.

All deficiencies resulting from material of production faults which occur during the period of warranty will be eliminated free of charge. Rights to claims under the terms of warranty are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of warranty.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of warranty, please return to the sales outlet in which you purchased the AMPLICOM equipment together with the purchase receipt. All rights to claims under the terms of warranty in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of warranty can no longer be asserted.

**Disposal** - The  symbol on this product indicates that electrical and electronic apparatus and battery pack must be disposed of **separately** from domestic waste at suitable collection points provided by the public waste authorities.

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