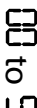
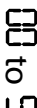
Display window icons

The following icons may appear in the display window:

---	(steady)	answering machine off
---	(flashing)	alternating with  or any other number: answering machine is on, day and time are not set
	(steady)	number of messages (already played)
	(flashing)	number of messages (not yet played)
R1	(steady)	recording outgoing message
FL		memory is full

Customer Service Information

Amplicom USA Service Center

310 Ambrose Avenue

Nashville, TN 37207

1-866-AMPLICOM (267-5426)


www.amplicomusa.com

Made in China

Answering Machine AB 900

AB 900 Answering Machine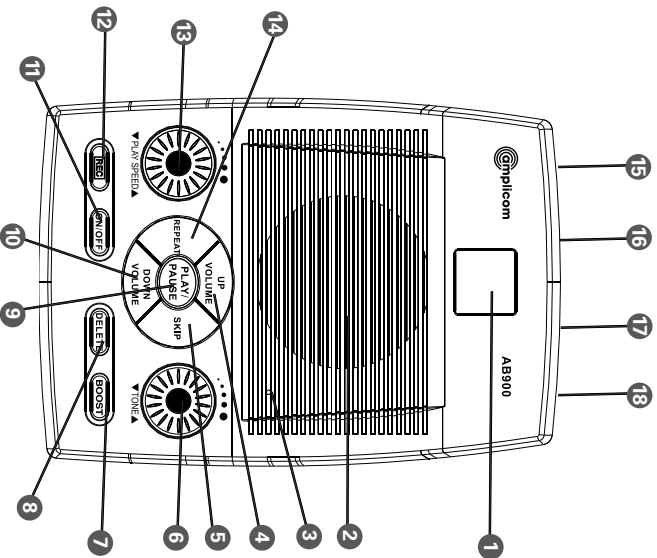

- 1 Display window
- 2 Speaker
- 3 Microphone
- 4 Volume up
- 5 Next message
- 6 Tone control
- 7 Boost on / off
- 8 Delete
- 9 Play / pause
- 10 Volume down
- 11 On / off / stop
- 12 Record
- 13 Speed control
- 14 Previous message
- 15 Power adapter socket
- 16 Headset jack
- 17 Ring delay switch
- 18 Telephone line jack

Warning: This machine has been specially designed for individuals who are hard of hearing and has a normal volume which is louder than a standard phone. Use of the Boost feature can cause hearing damage to individuals who are not experiencing hearing loss.

Set up

Connect answering machine

1. Plug the power adapter into the back of the answering machine and the other end into a wall socket. Turn the power on.
2. Plug the telephone cable into the back of the TEL. LINE jack on the answering machine and the other end into the phone. If your phone does not have a second jack, a line splitter may be necessary.

Record a custom greeting

1. Press and hold the **[RECORD]** button until the recording tone sounds. Continue holding down the button.
2. Start the greeting immediately after hearing the tone.
3. Release the **[RECORD]** button when finished recording.

Listen to messages

1. To listen to messages, press the **[PLAY/PAUSE]** button once.
2. Press the **[PLAY/PAUSE]** button while messages are playing to pause the playback.

3. Stop listening to messages at any time by pressing the **[STOP]** button.

Repeat the current message

Press the **[REPEAT]** button to repeat the message currently playing.

Repeat the previous message

Press and hold the **[REPEAT]** button until a signal tone is played to repeat the previous message.

Skip one message

Press the **[SKIP]** button once to skip the message currently playing. The unit will play the next message.

Fast forward

Press the **[SKIP]** button several times in succession to rapidly skip several messages.

Delete one message

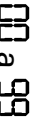
Press the **[DELETE]** button while a message is playing to delete the message.

Delete all messages

After the last message is played, press the **[DELETE]** button and hold until the unit plays a signal tone.

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